

CURA AIR CHAIR



CURA AIR CHAIR



Chair Position

We at CURA SEATING would like to thank you for choosing the CURA AIR CHAIR.

We hope that you enjoy the comfort and benefits that your chair has to offer.

Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Legrest



Raise



Lower



Tilt in Space



Tilt



Return



Backrest Recline

Lower



Raise





Adjustable Seat Depth

(First apply the brakes)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth.



Wing Handles

The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.

Charging the Battery

1. Take the charger lead and insert it into the charger socket, which is located at the back of the chair.
2. An overnight charge should give you around 100 movements.
3. If the battery is low a beep sound will let you know that it needs charging.
4. Always make sure that the chair has plenty of room behind for reclining.



Removable Pressure Relief

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down.

Lift up the velcro and then remove the cushion. The cushion is also velcroed down.

You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.




Directional Castor

How to turn the directional castor on.

Press the green brake on the castor and push the chair forward, the castor will click into place once the castor has centralised itself.




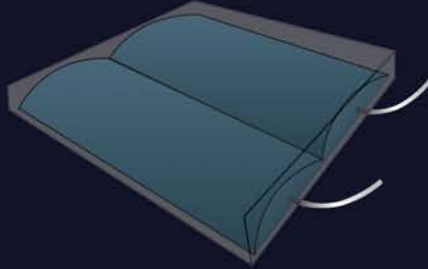

How to inflate the air cushion in the seat

There are two air cells, one across the front half of the cushion and one across the back half of the cushion. To inflate the air cell make sure that the client is not in the chair

1. Remove the air cap with an anti-clockwise turn.
2. Remove any existing air in the cushion until it is completely deflated by pushing down on the

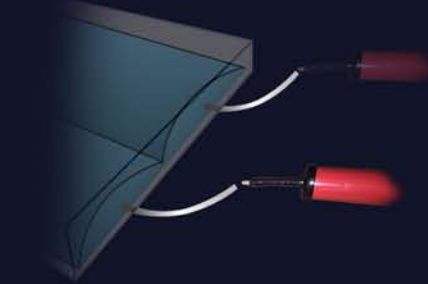

3. Insert the air pump nozzle into the air pipe and pump 20 times into the front and rear pipes .
4. Screw the cap back on as soon as you have inflated each air cell.



Removable Footplate

Fitting and removing the 90 degree angle footplate

1. Elevate the legrest to its max height so you can get access to the underside of the legrest.
2. You will find a black pull pin, pull the pin down to unlock and insert the footplate into the bracket located under the legrest.
3. To remove the footplate pull the pull pin and slide out the footplate
4. **Important** Always remove the footplate first before returning the chair to its upright position with the leg rest in the closed position as this could drag on the floor and would void your warranty.



CURA AIR CHAIR



CURA AIR CHAIR



Chair Position

We at CURA SEATING would like to thank you for choosing the CURA AIR CHAIR.

We hope that you enjoy the comfort and benefits that your chair has to offer.

Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Legrest



Tilt in Space



Backrest Recline

To set the back angle, hold the top rear handle and support it. Then Pull the plastic pull-pin which will then release the angle bracket and select the angle you require and release the pull-pin. This will automatically lock into place.





Adjustable Seat Depth

(First apply the brakes)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth.



Wing Handles

The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.



Charging the Battery

1. Take the charger lead and insert it into the charger socket, which is located at the back of the chair.
2. An overnight charge should give you around 100 movements.
3. If the battery is low a beep sound will let you know that it needs charging.
4. Always make sure that the chair has plenty of room behind for reclining.



Removable Pressure Relief

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down.

Lift up the velcro and then remove the cushion. The cushion is also velcroed down.


You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.



Directional Castor

How to turn the directional castor on.

Press the green brake on the castor and push the chair forward, the castor will click into place once the castor has centralised itself.

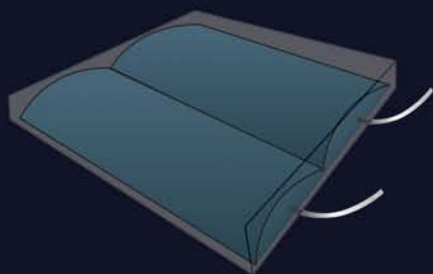


How to inflate the air cushion in the seat

There are two air cells, one across the front half of the cushion and one across the back half of the cushion. To inflate the air cell make sure that the client is not in the chair

1. Remove the air cap with an anti-clockwise turn.
2. Remove any existing air in the cushion until it is completely deflated by pushing down on the



3. Insert the air pump nozzle into the air pipe and pump 20 times into the front and rear pipes .
4. Screw the cap back on as soon as you have inflated each air cell.

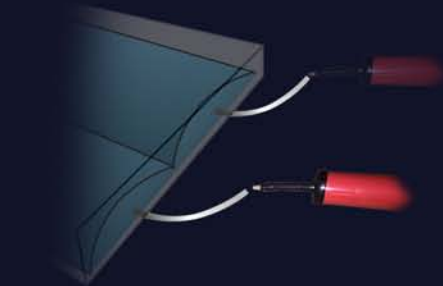

Removable Footplate

Fitting and removing the 90 degree angle footplate

1. Elevate the legrest to its max height so you can get access to the underside of the legrest.
2. You will find a black pull pin, pull the pin down to unlock and insert the footplate into the bracket located under the legrest.
3. To remove the footplate pull the pull pin and slide out the footplate
4. **Important** Always remove the footplate first before returning the chair to its upright position with the leg rest in the closed position as this could drag on the floor and would void your warranty.

