

CURA Attollo



CURA Attollo



Chair Position

We at CURA SEATING would like to thank you for choosing the CURA Attollo. We hope that you enjoy the comfort and benefits that your chair has to offer. Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Lift to Stand



Legrest



Tilt in Space



Backrest Recline





Adjustable Seat Depth (Option)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth. You will here the back rest click into place.



Horizontal Lift to Stand



Vertical Lift to Stand Option

To elevate the chair in a flat non tilting lift movement, press the two top buttons on the lower paired buttons as illustrated. To lower press the lower pair of the handset as illustrated.



Vertical Lift



Press to Lower



Horizontaly Raised + Legrest Position for Leg Dressings

Raised Horizontal Position for Carers



Elevated Flat Bed Platform Option

To make the chair go into an elevated flattened high platform, for carer use

- 1, Press the two top buttons on the lower paired buttons as illustrated.
- 2, Elevate the legrest.
- 3, Lower the backrest.

CURA SEATING



Raise the Arm at a Touch of a Button

Raise



Lower



Electric Lift Up Arm (Option)

When transferring from wheelchair into the chair have the top of the chair's seat surface 1" Lower.

When transferring from the chair into the wheelchair have the top of the wheelchair's seat surface 1" Lower.

0870 242 1358

www.curaseating.com

