

CURA LEGACY





Chair Position

We at **CURA SEATING** would like to thank you for choosing the **CURA LEGACY**.

We hope that you enjoy the comfort and benefits that your chair has to offer.

Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Legrest

Pull the lever to release the legrest, the leg rest will elevate.

Depending on the clients leg weight some assistance may be needed to help elevate the legrest.



Raise



Lower

Pull the lever to release the legrest and push down the leg rest.



Tilt in Space

Pull the brake lever to release the Tilt in Space gas strut and tilt the client towards you. As this is a manual chair you will need to help control the movement and push down the handles.

To bring the chair back up again release the brake lever and tilt the chair forward helping to control the movement.



Backrest Recline

To set the back angle, hold the top rear handle and support it. Then Pull the plastic pull-pin which will then release the angle bracket and select the angle you require and release the pull-pin. This will automatically lock into place.





Adjustable Seat Depth

(First apply the brakes)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth.



Adjustable Seat Width

The seat width can be changed by pulling the Pull Pin under the seat of the chair. Pull the Black plastic pin, this will release the arm allowing it to move inwards and outwards. Seat width are 16", 18" and 20".

Wing Handles



The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.



Changing the Arm to Seat Height

This can be done by pulling the cable underneath the seatbase from the back of the chair. Once you pull the cable this will then release the seat allowing you to lift up to the top hole which will give you 7", the middle hole 8" and the bottom hole 9". This can only be done when a person is not in the chair.



Removable Pressure Relief

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down. Lift up the velcro and then remove the cushion. The cushion is also velcroed down. You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.



Directional Castor

How to turn the directional castor on. Press the green brake on the castor and push the chair forward, the castor will click into place once the castor has centralised itself.

CURA SEATING




Next Generation Legacy Footplate

PAT: GB1720854.7


To operate the NGL Footplate

1. Elevate the leg rest.

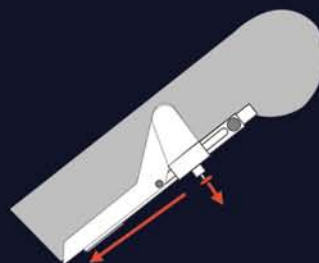


To Release the Footplate

2. Pull the underside pin to release the lock, then pull and slide the footplate down until it stops.

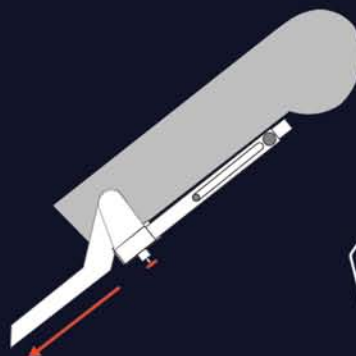

To Angle the Footplate

3. Pull the pull-pin on the side to release the lock, then rotate the footplate to a ninety, or selected, degree and release the pul-pin.

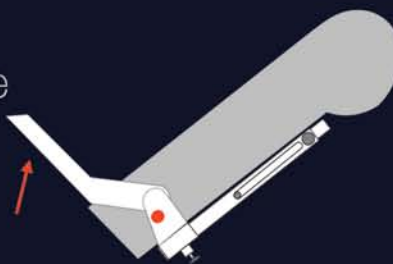

How to adjust the Footplate to seat height

4. Pull the under side pin to release the lock, which will allow you to extend the Footplate by 1" intervals



To put the footplate away,

5. Reverse the procedure.



0870 242 1358

www.curaseating.com



CURA LEGACY





Chair Position

We at **CURA SEATING** would like to thank you for choosing the **CURA LEGACY**.

We hope that you enjoy the comfort and benefits that your chair has to offer.

Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Legrest

Pull the lever to release the legrest, the leg rest will elevate.

Depending on the clients leg weight some assistance may be needed to help elevate the legrest.



Raise



Lower

Pull the lever to release the legrest and push down the leg rest.



Tilt in Space

Pull the Right Hand brake lever to release the Tilt in Space gas strut and tilt the client towards you. As this is a manual chair you will need to help control the movement and push down the handles.

To bring the chair back up again release the brake lever and tilt the chair forward helping to control the movement.



Backrest Recline

Pull the Left Hand brake lever to release the Backrest gas strut and recline the Backrest towards you. As this is a manual chair you will need to help control the movement and push down the handles.

To bring the chair Backrest up again release the brake lever and lift upwards helping to control the movement.





Adjustable Seat Depth

(First apply the brakes)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth.



Adjustable Seat Width

The seat width can be changed by pulling the Pull Pin under the seat of the chair. Pull the Black plastic pin, this will release the arm allowing it to move inwards and outwards. Seat width are 16", 18" and 20".

Wing Handles



The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.



Changing the Arm to Seat Height

This can be done by pulling the cable underneath the seatbase from the back of the chair. Once you pull the cable this will then release the seat allowing you to lift up to the top hole which will give you 7", the middle hole 8" and the bottom hole 9". This can only be done when a person is not in the chair.



Removable Pressure Relief

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down. Lift up the velcro and then remove the cushion. The cushion is also velcroed down. You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.



Directional Castor

How to turn the directional castor on. Press the green brake on the castor and push the chair forward, the castor will click into place once the castor has centralised itself.

CURA SEATING




Next Generation Legacy Footplate

PAT: GB1720854.7


To operate the NGL Footplate

1. Elevate the leg rest.

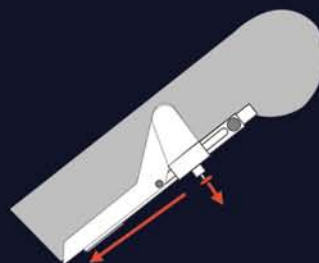


To Release the Footplate

2. Pull the underside pin to release the lock, then pull and slide the footplate down until it stops.

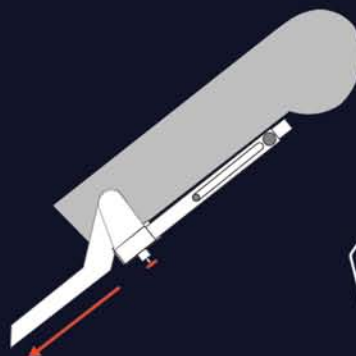

To Angle the Footplate

3. Pull the pull-pin on the side to release the lock, then rotate the footplate to a ninety, or selected, degree and release the pul-pin.

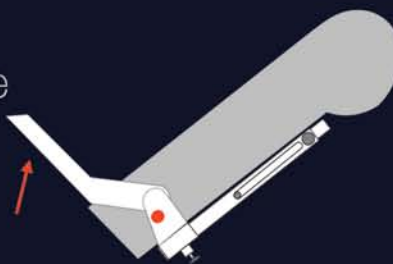

How to adjust the Footplate to seat height

4. Pull the under side pin to release the lock, which will allow you to extend the Footplate by 1" intervals




To put the footplate away,

5. Reverse the procedure.



0870 242 1358

www.curaseating.com

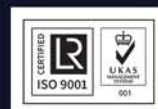

CURA LEGACY





Chair Position

We at **CURA SEATING** would like to thank you for choosing the **CURA LEGACY**.

We hope that you enjoy the comfort and benefits that your chair has to offer.

Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Legrest



Lower



Raise



Tilt in Space



Tilt



Return



Backrest Recline

To set the back angle, hold the top rear handle and support it. Then Pull the plastic pull-pin which will then release the angle bracket and select the angle you require and release the pull-pin. This will automatically lock into place.





Adjustable Seat Depth

(First apply the brakes)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth.



Adjustable Seat Width

The seat width can be changed by pulling the Pull Pin under the seat of the chair.

Pull the Black plastic pin, this will release the arm allowing it to move inwards and outwards.

Seat width are 16", 18" and 20".

Wing Handles



The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.

Changing the Arm to Seat Height

This can be done by pulling the cable underneath the seatbase from the back of the chair.

Once you pull the cable this will then release the seat allowing you to lift up to the top hole which will give you 7", the middle hole 8" and the bottom hole 9".

This can only be done when a person is not in the chair.

Removable Pressure Relief

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down.

Lift up the velcro and then remove the cushion. The cushion is also velcroed down.

You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.

Charging the Battery

1. Take the charger lead and insert it into the charger socket, which is located at the back of the chair.

2. An overnight charge should give you around 100 movements.

3. If the battery is low a beep sound will let you know that it needs charging.

4. Always make sure that the chair has plenty of room behind for reclining.



CURA SEATING




Next Generation Legacy Footplate

PAT: GB1720854.7


To operate the NGL Footplate

1. Elevate the leg rest.

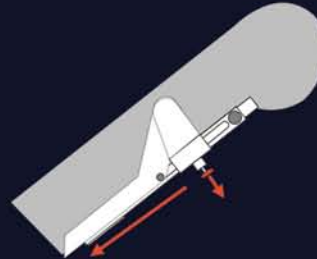



To Release the Footplate

2. Pull the underside pin to release the lock, then pull and slide the footplate down until it stops.

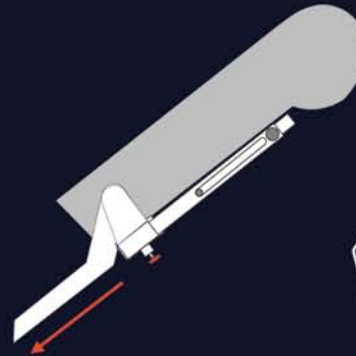

To Angle the Footplate

3. Pull the pull-pin on the side to release the lock, then rotate the footplate to a ninety, or selected, degree and release the pul-pin.

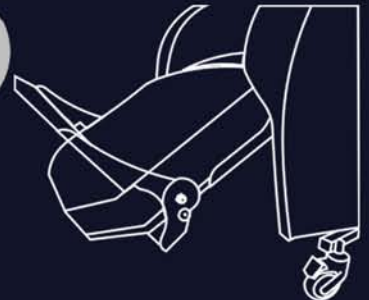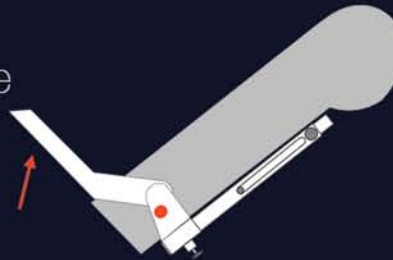

How to adjust the Footplate to seat height

4. Pull the under side pin to release the lock, which will allow you to extend the Footplate by 1" intervals



To put the footplate away,

5. Reverse the procedure.



0870 242 1358

www.curaseating.com



CURA LEGACY



CURA LEGACY



Chair Position

We at **CURA SEATING** would like to thank you for choosing the **CURA LEGACY**.
We hope that you enjoy the comfort and benefits that your chair has to offer.
Please follow the instructions to see how to operate the chair.
You will find in the blue sections how to adjust the chair sizes and features.



Legrest



Raise



Lower



Tilt in Space



Tilt



Return



Backrest Recline

Lower



Raise





Adjustable Seat Depth

(First apply the brakes)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth.



Adjustable Seat Width

The seat width can be changed by pulling the Pull Pin under the seat of the chair.

Pull the Black plastic pin, this will release the arm allowing it to move inwards and outwards.

Seat width are 16", 18" and 20".

Wing Handles



The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.

Changing the Arm to Seat Height

This can be done by pulling the cable underneath the seatbase from the back of the chair.

Once you pull the cable this will then release the seat allowing you to lift up to the top hole which will give you 7", the middle hole 8" and the bottom hole 9".

This can only be done when a person is not in the chair.

Removable Pressure Relief

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down.

Lift up the velcro and then remove the cushion. The cushion is also velcroed down.

You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.

Charging the Battery

1. Take the charger lead and insert it into the charger socket, which is located at the back of the chair.

2. An overnight charge should give you around 100 movements.

3. If the battery is low a beep sound will let you know that it needs charging.

4. Always make sure that the chair has plenty of room behind for reclining.



CURA SEATING




Next Generation Legacy Footplate

PAT: GB1720854.7


To operate the NGL Footplate

1. Elevate the leg rest.

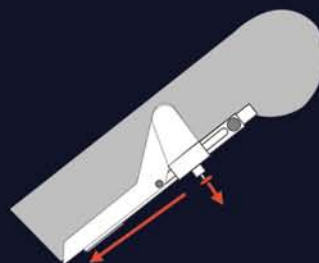


To Release the Footplate

2. Pull the underside pin to release the lock, then pull and slide the footplate down until it stops.

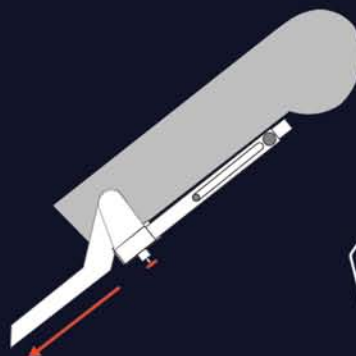

To Angle the Footplate

3. Pull the pull-pin on the side to release the lock, then rotate the footplate to a ninety, or selected, degree and release the pul-pin.

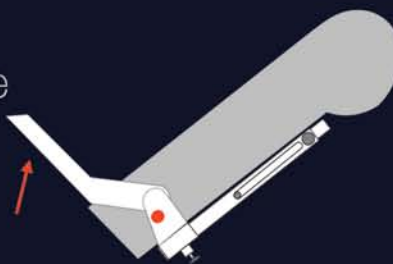

How to adjust the Footplate to seat height

4. Pull the under side pin to release the lock, which will allow you to extend the Footplate by 1" intervals



To put the footplate away,

5. Reverse the procedure.



0870 242 1358

www.curaseating.com

