

CURA Porta



CURA Porta



Chair Position

We at CURA SEATING would like to thank you for choosing the CURA Porta. We hope that you enjoy the comfort and benefits that your chair has to offer. Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Lift to Stand



Raise



Lower



Legrest



Raise



Lower



Tilt in Space



Raise



Lower



Backrest Recline



Lower



Raise



Adjustable Seat Depth (Option)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth. You will here the back rest click into place.

Removable Pressure Relief (Option)

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down. Lift up the velcro and then remove the cushion. The cushion is also velcroed down. You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.

Battery Powered

1. Take the charger lead and insert it into the charger socket, which is located at the back of the chair.
2. An overnight charge should give you around 100 movements.
3. If the battery is low a beep sound will let you know that it needs charging.
4. Always make sure that the chair has plenty of room behind for reclining.

Porta Wing Handles

The handles in the wings are a great benefit for pushing the chair when it is in a tilt in space position. This is because when the chair is tilted the rear push handle is too low to push someone.

Removable Footplate

- Fitting and removing the 90 degree angle footplate
1. Elevate the legrest to its max height so you can get access to the underside of the legrest.
 2. You will find a black pull pin, pull the pin down to unlock and insert the footplate into the bracket located under the legrest.
 3. To remove the footplate pull the pull pin and slide out the footplate
 4. **Important** Always remove the footplate first before returning the chair to its upright position with the legrest in the closed position as this could drag on the floor and would void your warranty.

CURA SEATING



Raise the Arm at a Touch of a Button

Raise



Lower



Electric Lift Up Arm (Option)

When transferring from wheelchair into the chair have the top of the chair's seat surface 1" Lower.

When transferring from the chair into the wheelchair have the top of the wheelchair's seat surface 1" Lower.

0870 242 1358

www.curaseating.com

