

# CURA

## Multi Action Rise & Recline



# CURA

## Multi Action Rise & Recline



Chair Position

We at CURA SEATING would like to thank you for choosing the CURA Multi Action Rise & Recliner. We hope that you enjoy the comfort and benefits that your chair has to offer. Please follow the instructions to see how to operate the chair. You will find in the blue sections how to adjust the chair sizes and features.



Lift to Stand



Legrest



Tilt in Space



Backrest Recline







## Adjustable Seat Depth (Option)

You can change the seat depth from 17" to 21". To change the seat depth, lift the velcroed down skirt up at the back of the chair and you will see the brake lever on the backrest bar. Hold the lower bar with your left hand and squeeze the brake lever bar with your right hand and push the lever bar forwards or pull it backwards to the persons required seat depth. You will here the back rest click into place.



## Removable Pressure Relief (Option)

To remove the pressure cushion lower the backrest to allow you to get access to the back of the overlay which is velcroed down. Lift up the velcro and then remove the cushion. The cushion is also velcroed down. You can insert another cushion if you wish and pull the overlay back down over the velcro at the back and bring the chair backrest back up.



## Electrical Angle of lift (Option)

To elevate the chair in a vertical non tilting lift movement, press the two top buttons as illustrated on the handset on the left.

To lower the chair, press the lower two buttons as illustrated on the handset on the right.

If you are having trouble operating this, we can offer an eight button handset, where one button can operate the functions instead of having to press two.



# CURA SEATING



Raise the Arm at a Touch of a Button

Raise



Lower



## Electric Lift Up Arm (Option)

When transferring from wheelchair into the chair have the top of the chair's seat surface 1" Lower.

When transferring from the chair into the wheelchair have the top of the wheelchair's seat surface 1" Lower.

0870 242 1358

[www.curaseating.com](http://www.curaseating.com)

